


<u>CoperSurgical</u>

Sound the Uterus

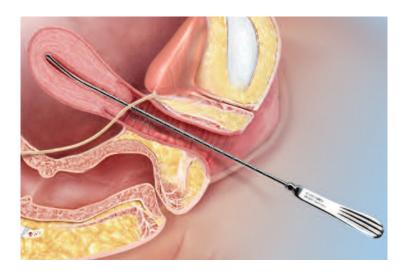

• The first step when doing any laparoscopic procedure utilizing uterine manipulation is to sound the uterus to measure length.


Choose the Correct Tip Size

A Full Range of Flexible Tips to Fit the Patient and the Procedure



- Choose the tip size that is just below the length of the uterus sounded.
- For example, choose an 8 cm tip for a uterus that is 9 cm in length.
- Using the correct tip reduces the risk of uterine perforation or trauma.

Place Tip on the Handle Easy Tip Application

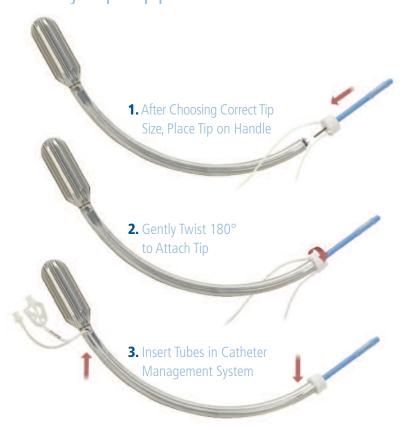

- Place tip on handle and gently twist 180° to lock in place.
- Insert tubes into catheter channels on either side of the handle. The catheter management system will improve site management during procedure.

Measure Cervical Size

Do's and Don'ts of Sizing



DO's


- With speculum in place, insert Cervical Sizer in a slightly closed position with numerical guide facing up.
- Once in vagina, open the sizer.
- Place Cervical Sizer blades up to fornix.
- Gently close Cervical Sizer until adjacent to the cervix.
- Read numerical value for proper Koh Cup[™] size.

DON'Ts

- Don't insert Cervical Sizer with numbers facing down.
- Don't stretch cervix with tenaculum during sizing.
- Don't remove Cervical Sizer prior to reading.
- Don't clamp Cervical Sizer down as it may cause undersizing of Koh Cup.

Preparing the Advincula Arch[™] for Patient Insertion with Koh-Efficient[®]

Three Simple Steps to Insertion

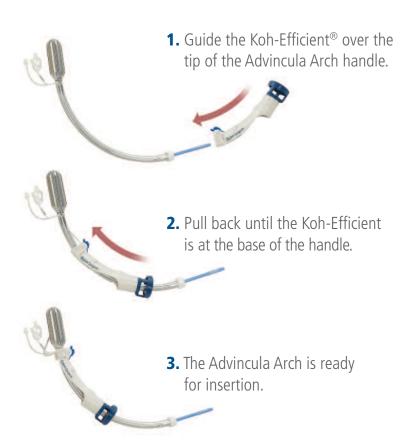

Insert Device into Uterus



Saline-inflated Tip Provides a Firm Hold for Manipulation



Koh-Efficient® is Easily Inserted Using Index Finger



Push Down to Lock

- The cervix should be dilated up to a size 8 using a Hank or Hegar dilator.
- Advance the uterine tip into the cervical canal.
- Inflate balloon tip with 3-5 cc's of saline.
- Gently push the Koh-Efficient forward using the index finger until it clicks audibly indicating it is locked in place.
- Once in place, engage the position lock by pushing down with your thumb over the full surface area of the lock.
 Typically, there is no need to use a suture or tenaculum when engaging the Koh Cup™ around cervix.
- Manually and visually check to ensure the cervix is completely seated within the Koh-Efficient.

Utilization of Manipulator



- Execute anteversion / retroversion manipulation of the uterus manually or with the Uterine Positioning System[™].
- Prior to starting the colpotomy, inflate the pneumo-occluder balloon with 60 cc's of saline.
- Cephalad pressure will help create a margin of safety for the colpotomy incision by increasing the distance between the ureters and the point of incision.



Scan. View. Learn.
Or go to coopersurgical.com/RUMI





Apply cephalad pressure to help create a Margin of Safety

REMOVE FROM SHAFI

How to Remove the Koh-Efficient®





Place Thumbs As Shown Above

Push Down to Release

- Disconnect the position lock.
- Unclip and remove locking mechanism from handle shaft.
- Slide the Koh-Efficient cup forward off of shaft to remove from device.
- Dispose of in accordance with all applicable Federal,
 State and local Medical/Hazardous waste practices.





CoperSurgical

CooperSurgical is the leading company dedicated to providing medical devices and procedure solutions that improve health care delivery to women regardless of clinical setting. Our company is fostering that position through expansion of its core businesses and introduction of advanced technology-based products, which aid clinicians in the management and treatment of commonly seen conditions.

203.601.5200 | 800.243.2974 | www.coopersurgical.com | 95 Corporate Dr. | Trumbull, CT 06611 Also on YouTube at www.YouTube.com/coopersurgical



Advincula Arch is engineered with our easy load system making it simple and easy to insert.

Scan. Watch. Learn.

or go to coopersurgical.com/AdvinculaArch

The marks bearing the symbols 'TM' and '®' are trademarks and registered trademarks of CooperSurgical, Inc.

©2014 CooperSurgical, Inc.