



# Procedure Guide

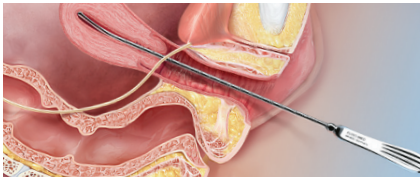

## 1. Sound the uterus.

The first step when doing any  $\emptyset$  procedure utilizing uterine manipulation is to sound the uterus to measure length.

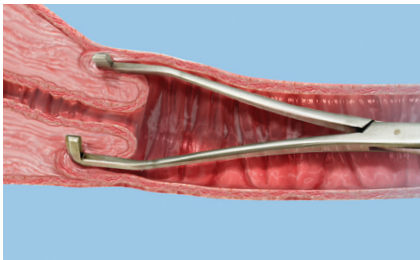


## 2. Measure cervical size.

- With speculum in place, insert Cervical Sizer in a slightly closed position with numerical guide facing up.
- Once in vagina, open the sizer.
- Place Cervical Sizer blades up to fornix.
- Gently close Cervical Sizer until adjacent to the cervix.
- Read numerical value for proper Koh Cup™ size.



## 3. Select the appropriate sized Delineator™ with Koh-Efficient®.

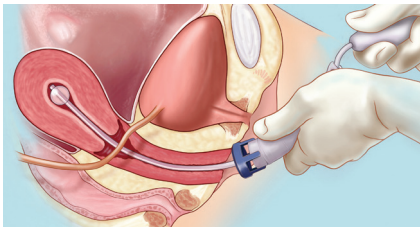


## 4. Place manipulator through cervix and inflate uterine balloon.

## 5. Slide the Koh-Efficient to the sounded number marked on the handle.



## 6. Press the position lock to secure the cup, and begin the procedure.

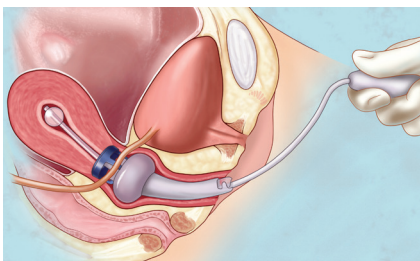

## 7. Utilization of manipulator.

- Execute manipulation of the uterus manually or with the ALLY Uterine Positioning System®.
- Prior to starting the colpotomy, inflate the pneumo-occluder balloon.
- With proper cephalad pressure, the Koh-Efficient helps to create a "Margin of Safety" by distancing the colpotomy incision from the ureters.