# Hoyte Sacro Tips

EN

LH-SCOLPO-SML, LH-SCOLPO-LRG, LH-SCERV-SML, LH-SCERV-LRG

## Instructions For Use

CE0086

Small  
Sacrocolpopexy Tip

Large  
Sacrocolpopexy Tip

Small  
Sacrocervicopexy Tip

Large  
Sacrocervicopexy Tip

### DEVICE DESCRIPTION

The Hoyte Sacrocolpopexy Tips (Hoyte Sacro Tips) are used for laparoscopic vaginal prolapse repair procedures where the cervix has been removed. The Hoyte Sacrocolpopexy Tip is made out of a hard polycarbonate plastic offered in a small and large tip size.

The Hoyte Sacrocervicopexy Tips (Hoyte Sacro Tips) are used for laparoscopic vaginal prolapse repair procedures where the cervix is intact. The Hoyte Sacrocervicopexy Tip is made out of a hard polycarbonate plastic offered in a small and large tip size.

### WARNINGS

Contents supplied sterile. Do not use if sterile barrier is damaged.

For single use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

Dispose of in accordance with all applicable Federal, State, and local Medical/Hazardous waste practices.

### CAUTION

U.S. Federal law restricts this device to sale by or on the order of a physician.

### INDICATIONS FOR USE

The Hoyte Sacro Tips are intended to be connected to the Advincula Arch™ handle (UMH750) and utilized for any laparoscopic or open procedure where elevating the vaginal tissue into the operative field would be beneficial.

The single use Hoyte Sacro Tips are intended to provide a solid base, or “thimble-like” surface to suture against.

**CONTRAINDICATIONS**

The Hoyte Sacro Tips should not be used when a patient has a non-patent vagina or vaginal septum.

**DIRECTIONS FOR USE**

Connecting the Hoyte Sacro Tip to the Advincula Arch™ handle (UMH750).

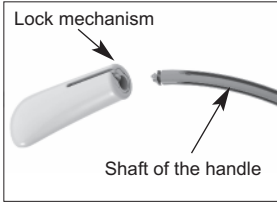
Fig. 1

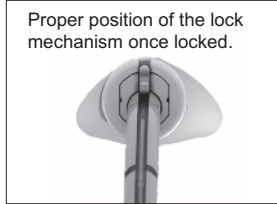
Fig. 2

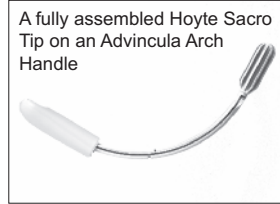
Fig. 3

1. Grasp the Hoyte Sacro Tip with the lock-mechanism facing toward the bottom of the shaft of the handle. (Fig.1)
2. Slide the Hoyte Sacro Tip over the shaft approximately 2 inches (5.08 cm) until it locks into place. (Fig. 2)
3. Attempt to pull back on the Hoyte Sacro Tip to confirm it is locked.

**EXPLANATION OF SYMBOLS**

**REF** Reorder number

**LOT** Batch code

 Use-by date

**STERILE EO** Sterilized using ethylene oxide

 Consult instructions for use

 Caution

 Not made with natural rubber latex

 Do not re-use

 Do not re-sterilize

 Do not use if package is damaged or opened

**R<sub>x</sub> Only** **Caution:** U.S. Federal law restricts this device to sale by or on the order of physician.

 Manufacturer

**EC REP** Authorized Representative in the European Community

**CE 0086** Product conforms to the Medical Device directive 93/42/EEC.

CooperSurgical is a registered trademark of CooperSurgical, Inc.  
Advincula Arch™ is a trademark of CooperSurgical, Inc.

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© 2015 CooperSurgical, Inc.
CooperSurgical, CooperSurgical, Inc. şirketinin tescilli ticari markasıdır.
Advincula Arch™, CooperSurgical, Inc. şirketinin tescilli ticari markasıdır.