


Hegenberger Retractor

Instructions for use


The Hegenberger Retractor is intended for women after a vaginal delivery. Only to be used by a clinical educated professional after understanding the instructions for use.

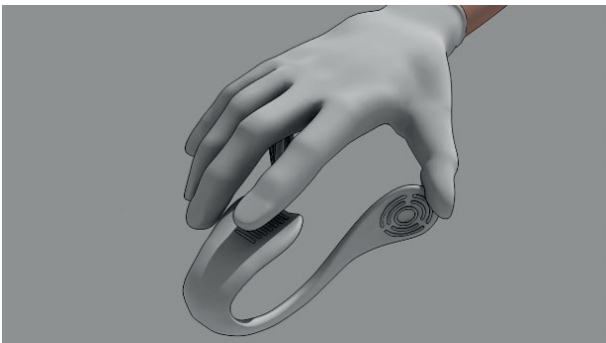


1 Open the pouch and remove the retractor. Hold the retractor as shown in the image.



2 Gently press the sides of the retractor together before inserting.

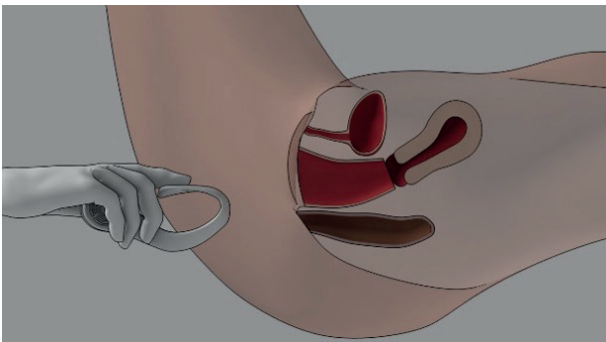


3 It is important that the patient is in a supine position with the legs elevated in stirrups and a well-functioning local anaesthetic.

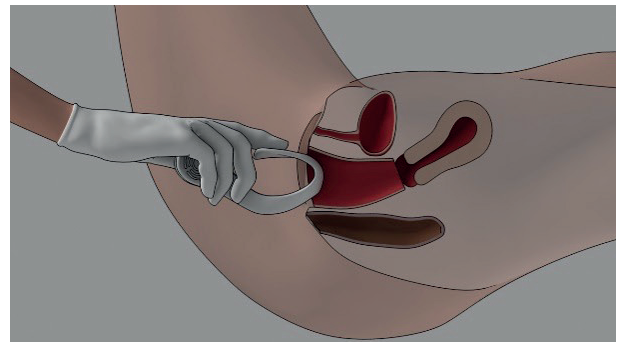


4 Keep the sides of the retractor compressed while inserting it in the vagina. **Do not apply lubrication.**



Instructions for use

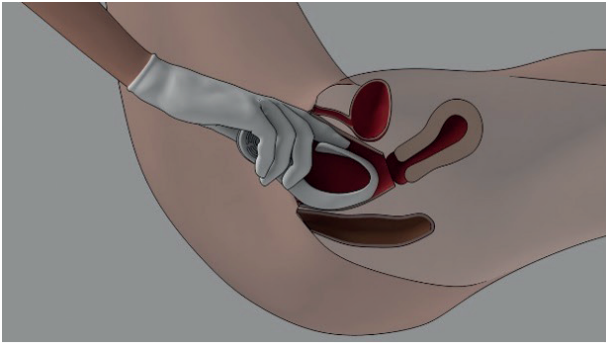


- 5 Ensure that the top of the retractor has been inserted past the pubic bone before releasing the sides.

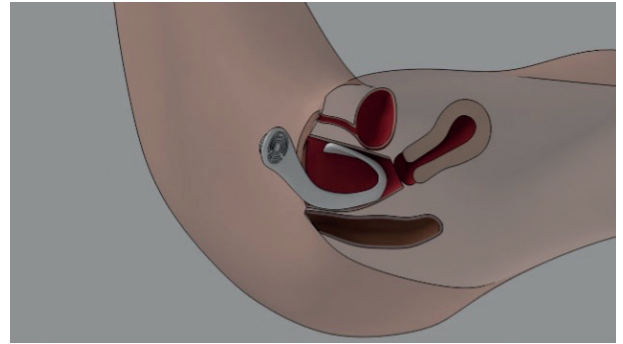

- 6 Correct placement is shown in this image. **Make sure the retractor is not placed further in than shown in this image.**

Removing the Speculum

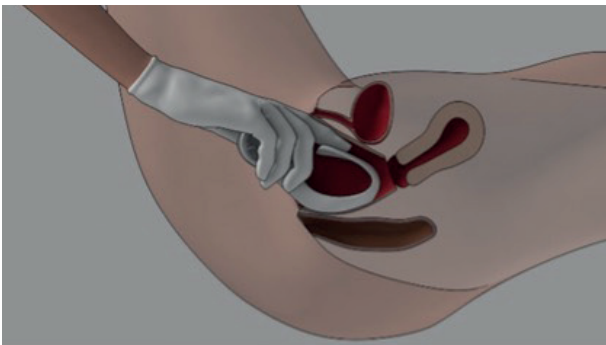

- 7 Ensure that the top of the retractor has been inserted past the pubic bone before releasing the sides.

Contraindications

The retractor is only intended for use on women following vaginal delivery. It is only intended for use by trained clinical professionals after understanding the instructions for use. Only to be used in clinical settings or hospitals. The patient must be in a supine position with elevated legs. Not to be used for pediatric patients or for placement in patients who have not delivered or in cases where force would be necessary for insertion. Not to be used in cases involving post-partum haemorrhage. Not to be used without informed consent from the patient. Not to be used during pregnancy. Not to be used for anterior wall procedures. Not for reuse.