GINRAM®

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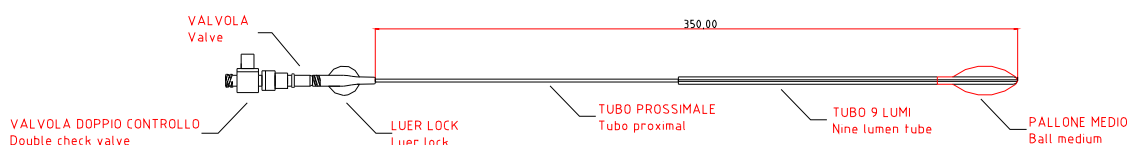
Instructions for use

Indications in Gynaecology:

Treatment and prevention of vaginal haemorrhage and haematoma after vaginal surgery or vaginal trauma.

Vagistop in Gynaecology replaces the use of vaginal packing with gauze.

Indications in Obstetrics:

Treatment of postpartum haemorrhages due to vaginal lacerations and prevention of postpartum vaginal hematomas.

Vagistop in Obstetrics replaces the use of vaginal packing with gauze.

Instructions:

- 1) Insert Vagistop up to the posterior fornix; be careful not to insert it into the uterus through the cervix.
- 2) Connect a syringe to the double control valve and inflate with air. (The 3-way valve keeps the syringe connected during the inflation procedure).
- 3) Inflate Vagistop until the balloon completely adheres to the vaginal walls and the lower end of the inflated balloon is visible through the vulvar ostium (opening) by slightly pulling on its stem. When Vagistop is inflated until it adheres to the vaginal walls, it maintains an internal pressure of approximately 50 mmHg, therefore greater than pelvic capillary pressure, which varies from 21 to 48 mmHg.

Recommended inflation volumes for Gynecological use: from 60 to 180 ml according to the size of the vagina.

Recommended inflation volumes for Obstetric use: from 240 to 480 ml according to the size of the vagina

The balloon has been tested up to a maximum air inflation of 500 ml.

- 4) Once the Vagistop has been positioned, remove the syringe with the double control valve (3-way valve).
- 5) In the case of Obstetric use, a bladder catheter should be used to control diuresis.
- 6) To remove the device, slowly deflate the balloon by pressing the button of the valve positioned at the base of the stem with a small syringe without plunger and remove Vagistop. Vagistop may be removed by the department personnel.

Warning: there may be a tendency for the balloon to slip outside, especially after vaginal surgery for prolapse in old age, caused by inelasticity and reduced size of the vagina. In selected cases, consider whether the application of steri strips is necessary to close the vulva commensurate.

Warning: Read the instructions before use.

Vagistop must not be left in place for more than 24 hours.

The application of Vagistop does not require the use of anaesthetics and does not require antibiotic prophylaxis.

After the application of Vagistop, continue to monitor the patient (diuresis, BP, etc.) to identify signs of any persistent vaginal bleeding.

Even after removal of Vagistop, check for any persistent bleeding.

Vagistop is not a substitute for surgical treatment when indicated.

Vagistop is not a substitute for treatments aimed at restoring blood volume (liquids, plasma, blood).

There is no clinical data in existence on the use of Vagistop in Disseminated Intravascular Coagulation (DIC).

Do not permit the second use. Use only if the packaging is intact. The device must be used immediately after opening the package. The device must only be used by specialised medical personnel.

Do not use the instrument in the following cases: DIC, existing vaginal infections, arterial bleeding that requires surgical treatment.

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