PATIENT INSTRUCTIONS INFLATOBALL™ PESSARY

Contains Latex Rubber

INSERTING • REMOVING • CLEANING THE PESSARY

CAUTION: This product contains natural rubber latex which may cause allergic reactions.

Latex Rubber absorbs secretions and odors.

The Inflatoball™ pessary has only a small area for drainage of vaginal secretion; it should therefore be removed and cleaned daily.

IMPORTANT

A pessary should not be worn in the presence of a vaginal infection. Should you notice a heavier than usual vaginal discharge with a foul odor, with or without vaginal itching or tenderness, remove pessary and call your health care professional promptly.

Empty bladder (urinate) before inserting pessary.

TO INSERT

1. Wash your hands thoroughly. Deflate pessary by moving the bead to closed end of the stem (see Figure 1). Apply light pressure with thumb and fingers to force out any air that may be in pessary (see Figure 2).
2. Keep pessary deflated by continuing light pressure with thumb and fingers.
3. Now insert metal part of bulb into the air vent (see Figure 3).

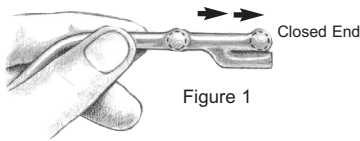
Figure 1

Figure 2

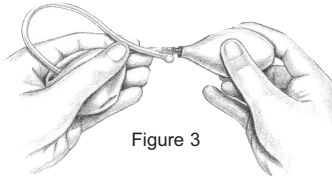
Figure 3

4. Hold the deflated pessary between the thumb and fingers for insertion. If necessary the leading edge of the INFLATOBALL may be lubricated with TRIMO-SAN™.

CAUTION: Do not use petroleum (vaseline), oil or wax based jelly or cream as they will cause the pessary to deteriorate.

5. Insert pessary into the vagina so that only the stem protrudes/hangs out from the vagina (see Figure 4).

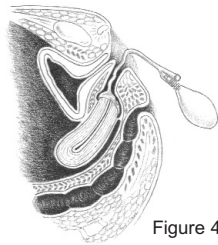
Figure 4

6. Inflate pessary by squeezing bulb (see Figure 5).

7. Push small bead located at the tip of the stem about 1 to 1½ inches forward. This will close the air vent and will keep your pessary inflated while it is worn (see Figure 6).

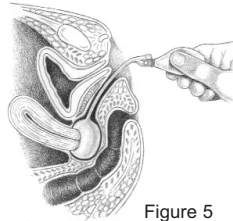
Figure 5

8. Remove the bulb from the air stem. The stem can be tucked into the vagina. The pessary should not move or be felt once it is in place and inflated.

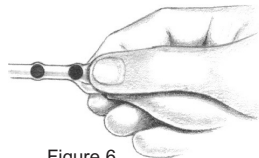
Figure 6

TO REMOVE PESSARY

Wash hands thoroughly.

1. Slip stem of pessary out of vagina.
2. Push bead down to its original position (closed end) to allow air to escape from the pessary (see Figure 1).
3. Now grasp the deflated pessary and withdraw. DO NOT pull on the pessary stem.

TO CLEAN PESSARY

1. Wash with mild soap and water.
2. Rinse thoroughly before reinserting.
3. Do not immerse bulb inflator in water.

PURPOSE OF TRIMO-SAN

1. To help restore and maintain normal vaginal acidity.
2. To coat the walls of the vagina with a lubricating film that helps to reduce odor-causing bacteria.

Instructions are included with each TRIMO-SAN package.

TRIMO-SAN is available upon request at the prescription counter of your pharmacy. No prescription is required.

FOLLOW-UP INSTRUCTIONS:

It is not unusual, after wearing a pessary, that it will be necessary to change the size or shape of the pessary being worn. This is why it is so important to follow this program:

- Report immediately any discomfort.
- Return within 24 hours for first examination.
- Return for second examination within three days.
- Return at four- to six-week intervals.

Note: Above schedule of follow-up examinations may be altered to fit the needs of the individual patient.

WARNING: The Milex™ INFLATOBALL Pessary is made of latex rubber. As a result of a MEDICAL ALERT issued by the FDA, March 29, 1991, on allergic reactions to latex-containing medical devices, the following recommendations are made:

- Remove pessary and call your health care professional if you have any symptoms of sensitivity to latex rubber such as: itching, rash, wheezing when wearing rubber gloves or inflating a balloon.

CONTRAINDICATIONS:

Supportive pessaries are contraindicated in presence of vaginal, cervical or pelvic infections or lacerations.

Important:

The Inflatoball (Latex Rubber) pessary should NOT be used in a patient using ANY vaginal hormone cream or suppository, including Premarin® cream or suppository.

Vaginal hormonal products contain a wax base which will deteriorate latex rubber. Use only TRIMO-SAN; other materials have not been tested for compatibility.

DO NOT OVER INFLATE:

Over inflation distorts and weakens latex. If your pessary “balloons” out, it is an indication it has been over inflated and you need a larger size Inflatoball or a different type of pessary.

No credit or replacement for pessary will be allowed because of over inflation or if used contrary to above restrictions.

For medical emergencies only

Consult your health care professional for all other medically-related advice.

CE0086

Premarin® is a registered trademark of Wyeth LLC.

Milex™, TRIMO-SAN™ and INFLATOBALL™ are trademarks of CooperSurgical, Inc.

CooperSurgical is a registered trademark of CooperSurgical, Inc.

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© 2012 CooperSurgical, Inc.
Premarin® est une marque déposée de Wyeth LLC.
Milex™, TRIMO-SAN™ et INFLATOBALL™ sont des marques de CooperSurgical, Inc.
CooperSurgical est une marque déposée de CooperSurgical, Inc.