

Hegenberger Speculum

We are helping obstetricians, surgeons, nurse midwives and midwives around the world to improve the ease, speed and quality of post-partum suturing with a revolutionary tool that provides a clear and unobstructed view and access while leaving the user's hands free to perform suturing. This saves time, reduces personnel requirements and costs and ensures a better overall patient experience.



**Optimal visualization
of the rupture**



Faster suturing



Better ergonomics


**For more
information
please contact:**

Malene Hegenberger,
Certified Nurse Midwife
Founder & CEO


Mail: malene@hegenbergerspeculum.com

Phone: +45 5354 4664


Web: www.hegenbergerspeculum.com

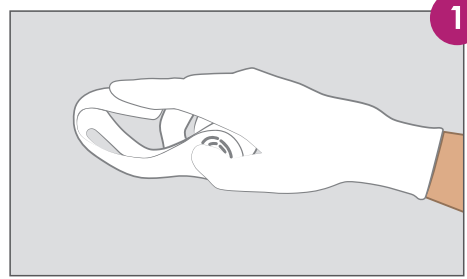


Hold the speculum as shown in the image.

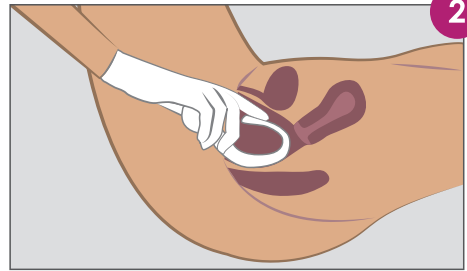


Keep the sides of the speculum compressed while inserting it in the vagina. **Do not apply lubrication.**

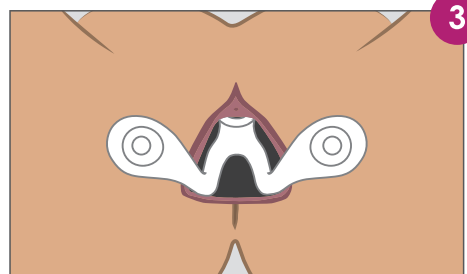

Correct placement is shown in this image. **Make sure the speculum is not placed further in than shown in this image.**

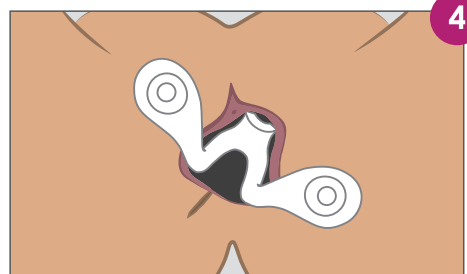

The speculum can be placed for lateral tears or episiotomies.

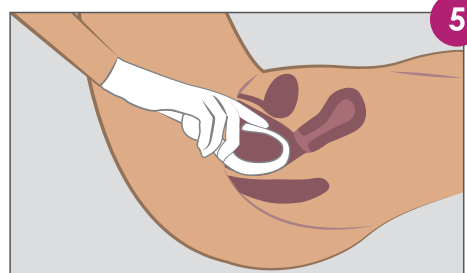

To remove the speculum:
Gently press the sides of the speculum together and use your index finger to ensure the top part of the speculum is clear of the cervix. Press down on the top part and gently retract the speculum from the vagina.

**HEGENBERGER
SPECULUM**

HEGENBERGER SPECULUM



FOR BETTER POST-PARTUM SUTURING

- Clear view of perineal trauma
- Made to improve quality of suturing
- Less patient stress due to less pain
- Reduces suturing time
- No need for a second pair of hands
- Improved training conditions
- Reduced risk of needlestick injury