

HOW DOES IT WORK?

1) BALLONS INFLATION

1° EXTERNAL BALLOON

60 ML syringe of sterile saline solution → BLUE LUMEN: "WATER"
(close the stopcock)



2° INTERNAL BALLOON

Same 60 ML syringe of air → TRANSPARENT LUMEN: "AIR INFLATION"
MAX 420 CC (CA 7 PUMPS CONTINUOUSLY)
(close the stopcock)



2) CONNECT THE DRAINAGE BAG




Fluid collection bag (included) → **RED LUMEN (DRAINAGE)**

3) CATHETER REMOVAL

1° INTERNAL BALLOON


TRANSPARENT LUMEN → Remove the valve and leak the air out

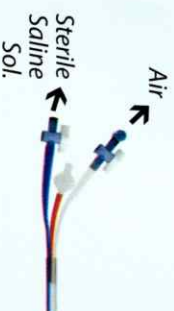

2° EXTERNAL BALLOON

BLUE LUMEN → Aspirate the content with a syringe

It is also possible to cut the catheter lumens in order to deflate the balloons more quickly -

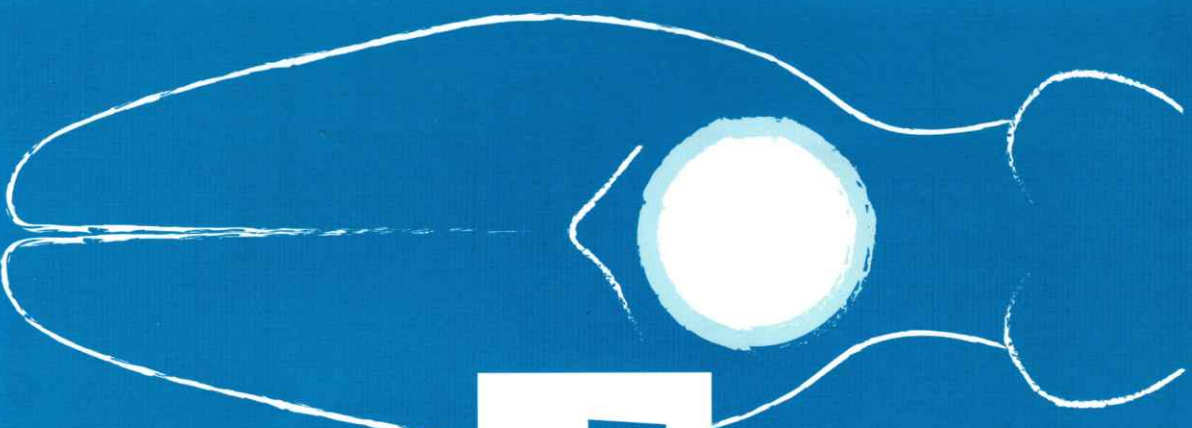

METRASTOP

Postpartum Balloon

MADE IN ITALY



RI.MOS.

Disposable Medical Products